



Street Electric Manufacturing Company LLC

Adaptive Technology • Software & Services

<http://www.quadjoy.com>
custserv@quadjoy.com
techsupport@quadjoy.com

3211 N. Koning Drive
Sheboygan, WI 53083

Phone: 1-877-736-2663
Fax: 920-694-0023

Cleaning and Hygienic Instructions

The Quadjoy Mouse has several hygienic features which must be adhered to strictly for the well being of both the user and the Quadjoy Mouse Unit itself. These features include quick change hygienic sticks and tubing with a clean out lure.

Hygienic Procedures must be performed under the following conditions:

1. Whenever moisture is observed in the clear tubing protruding from the side of the box.
2. When changing users.
3. When sipping and puffing becomes erratic or hard to use.

The hygienic and cleaning instructions for the Quadjoy Mouse are as follows and must be followed closely:

First remove the hygienic stick from the Quadjoy by firmly holding the square base of the stick with your thumb and index (pointer) finger of your left hand, then with the thumb and index finger of your other hand firmly grasp the semi-transparent lure-fitting at the base of the hygienic stick. While still holding the base firmly with you left hand, twist counterclockwise on the lure fitting with you right hand. As you are twisting the stick counter clockwise pull slightly away from the Quadjoy Mouse with your right hand. This will remove the hygienic stick from the Quadjoy Mouse unit. Then swab the inside of the stick base with a q-tip with rubbing alcohol, followed by a dry q-tip. After swabbing the inside of the stick base, you must remove the clear hygienic tubing from the stick base. To do this you will need to once again grasp the square part of the stick base with your thumb and index finger on your left hand. With you right hand grasp the clear tubing as close to the square part of the stick base as possible and pull straight outwards and away from the square part of the stick base. Once you have removed the plastic tubing from the stick base you will need to remove it from the Quadjoy Mouse box. To do this you will need to grasp the semi-transparent lure fitting attached to the other end of the clear tubing with your thumb and index finger on your right hand while holding the Quadjoy Mouse with your left hand. Twist the semi-transparent lure fitting counterclockwise with your right hand until it separates from the Quadjoy Mouse box. You will then want to replace all parts removed with cleaned or new hygiene parts from SEMCO. Installation is the reverse of removal.

If you have any questions during the setup or cleaning process please call Quadjoy Customer Support between 9:00 AM and 5:00 PM CST at 1-877-736-2663.

Thank You!